

NESCO® All New MultiCheck®

Glucose
Cholesterol
Uric Acid
Hemoglobin



SETTING TIME & DATE

The meter will set mode automatically when you insert batteries. Please set correct time and date before you begin testing.

1

Slide battery cover off and insert one LITHIUM CELL (CR2032, 3V) into the battery compartment.



2 The meter will turn on automatically.



The year will flash.
Press 『S』 button to set the correct year.

The month will flash.
Press 『S』 button to set the correct month.

3

Press 『M』 button to move to the date setting.
The date will flash.
Press 『S』 button to set the correct date.



4

Press 『M』 button to move to the hour.
The hour will flash.
Press 『S』 button to set the correct hour.



5

Press 『M』 button to move to the minute.
The minute will flash.
Press 『S』 button to set the correct minute.



6

The meter will automatically turn off after 30 seconds later, while "minute" is set.