


NESCO® All New MultiCheck®

Glucose
Cholesterol
Uric Acid
Hemoglobin

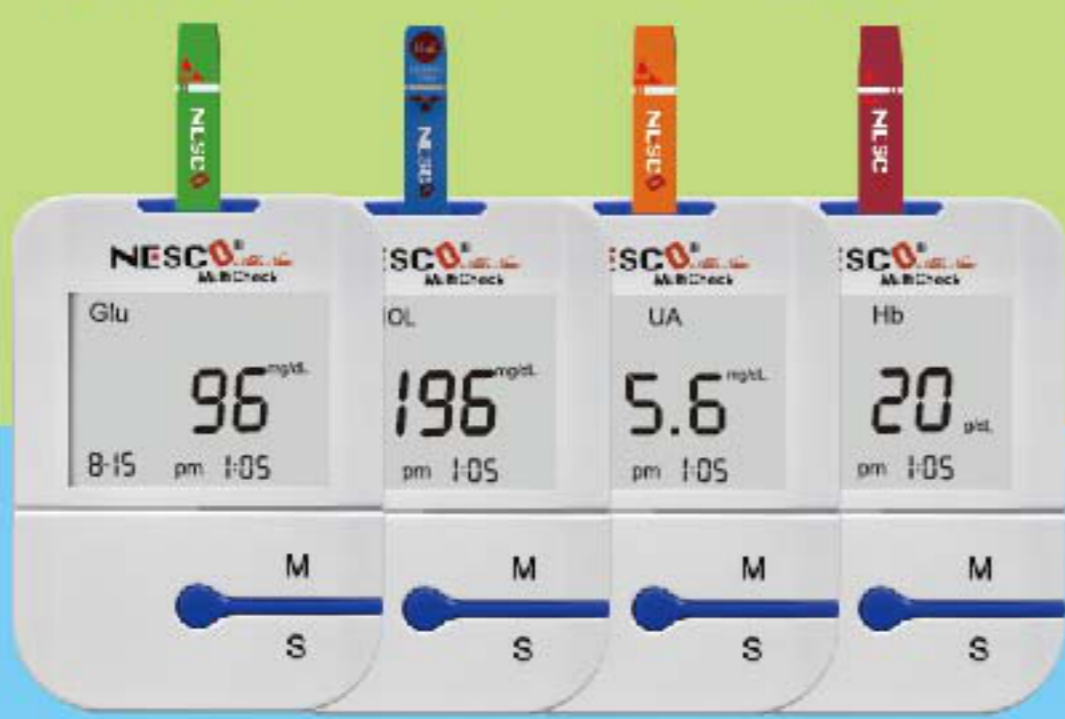

Glucose Glucose Testing Steps

- 8 Check Code Number
- 9 Insert Code Card
- 10 Turn On Automatically Insert Test Strip
- 11
- 12 Absorption
- 13 10 Seconds Display Result

Cholesterol Cholesterol Testing Steps

- 8 Check Code Number
- 9 Insert Code Card
- 10 Turn On Automatically Insert Test Strip
- 11 Collect Blood Sample
- 12 Absorption
- 13 150 Seconds Display Result

Uric Acid Uric Acid Testing Steps

- 8 Check Code Number
- 9 Insert Code Card
- 10 Turn On Automatically Insert Test Strip
- 11 Collect Blood Sample
- 12 Absorption
- 13 20 Seconds Display Result

Hemoglobin Hemoglobin Testing Steps

- 8 Check Code Number
- 9 Insert Code Card
- 10 Turn On Automatically Insert Test Strip
- 11 Collect Blood Sample
- 12 Absorption
- 13 6 Seconds Display Result

- 1 Unscrew Puncturer's Tip
- 2 Insert Lancet
- 3 Unscrew Protective Cover
- 4 Screw Puncturer's Tip
- 5 Adjust the Best Depth of Skin Penetration
- 6 Pull Up the Sliding Barrel
- 7 Hear "Click" Sound Release the Sleeve
- 14 Pull the sliding barrel again
- 15 Push the sliding strip to remove the lancet